

Caring for Your M7 SpeedClave® Sterilizer



WARNING

Make sure the Sterilizer is cool before attempting to clean to prevent personal injury from burns.



EQUIPMENT ALERT

Never use abrasive or bleaching agents (steel wool, scouring powder, bleach, etc. or a wire brush) to clean chamber. Damage to the chamber or related components could result.

DAILY

1. External Surfaces


Wash with damp cloth and mild soap solution, then wipe with soft, dry cloth.

2. Door Gasket & Surface

Wipe door gasket & mating surface with a damp cloth.

3. Door Gasket

Examine gasket for damage and replace if necessary.

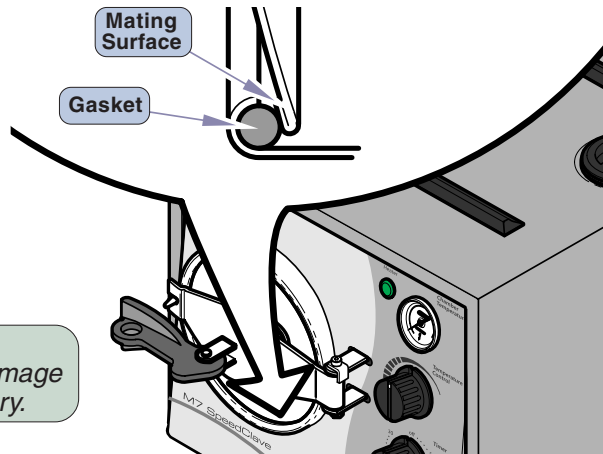

WEEKLY

Note


Refer to "Tray Rack" for Removal & Installation Instructions.

1. Trays & Chamber

Wash with damp cloth and mild soap solution, then wipe with soft, dry cloth.

2. Drain Reservoir

Place end of hose in container and connect hose adapter to coupling.

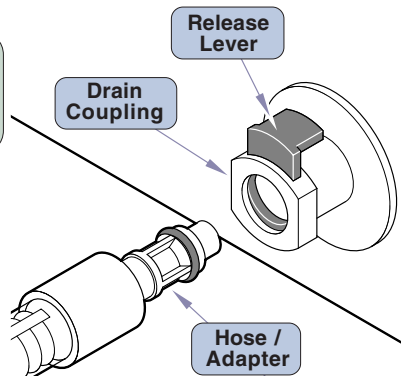

3. Remove Hose

Press down on release lever and remove hose assembly.

Note

If drain coupling leaks after insertion or removal reinsert adapter / hose several times to clean seals, stopping leak.


4. Fill Reservoir

Use only distilled or demineralized water.

MONTHLY

Note

Never sterilize instruments while performing cleaning procedures.

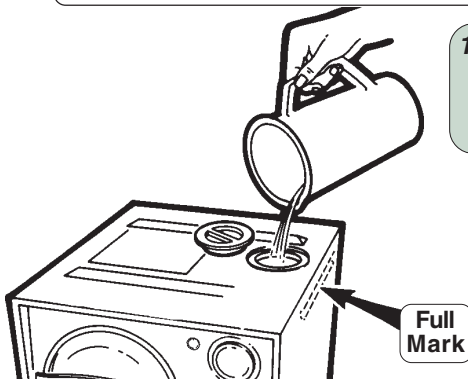


1. Drain & Refill Reservoir

Drain reservoir and refill with clean distilled or demineralized water. (Refer to Weekly Cleaning)

2. Add Cleaner

Add 1 oz. (29.6 ml) of Speed-Clean to a cool chamber.

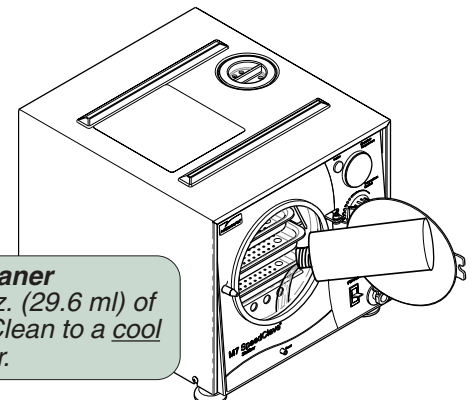

MONTHLY (continued)

3. Fill Chamber

Press down and hold Fill / Vent Lever until water is within 1/2" (1.3 cm) of rim.

4. Run Cycle

Run one 6 Minute cycle at 270°F (132°C).

5. Drain & Refill Reservoir

Drain chamber and reservoir and refill with clean distilled or demineralized water. (Refer to Weekly Cleaning)

6. Run Cycle


Run one 3 Minute cycle at 270°F (132°C).

7. Drain & Cool

Drain chamber and reservoir and allow sterilizer to cool. Remove trays and tray rack. (Refer to Tray Rack Removal)


8. Clean Chamber, Filter, & Trays

Remove and clean chamber filter, wipe out chamber and clean trays and tray rack.

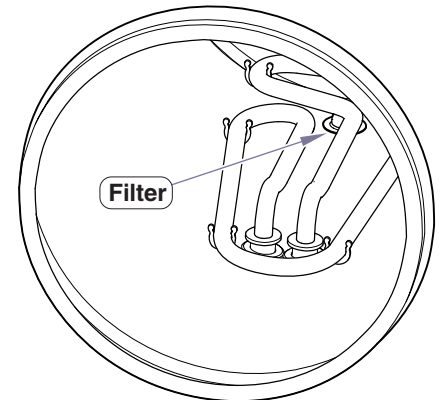

9. Refill Reservoir

Refill reservoir with clean distilled or demineralized water. Install chamber filter, tray rack and trays.

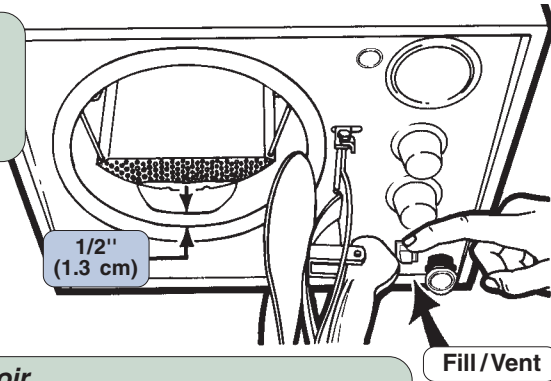

TRAY RACK



WARNING

Make sure unit is cool before attempting to remove or install tray rack and plate. Use care to prevent injury when handling metal tray rack assembly. Do **NOT** run sterilizer without tray rack assembly in place.

1. Removal

Lift up on left edge of tray plate until it "pops" free of wire rack. Slide tray out of chamber.

2. Removal

Squeeze bottom of wire rack together and slide out of chamber.

1. Installation

With the offset ends of the rack on the left, gently squeeze bottom of rack together and insert into chamber as far as it will go.

2. Installation

Holding tray plate with right side point down, insert into chamber positioning right side of tray plate under bottom rack wire aligning straight ends of rack with notches on plate.

3. Installation

Holding right side of plate in position, press down left side of plate until it snaps into offsets of wire rack.

