


### Recommended Installation

When possible, the Anprolene Sterilizer should be located near an outside wall, and exhaust directly to the outside by means of  $\frac{3}{4}$ " (1.9cm) I.D. tubing. Through wall mounting kits are available from Andersen. Note: Sterilizer must be placed in an area that can maintain 68°F or above room temperature.

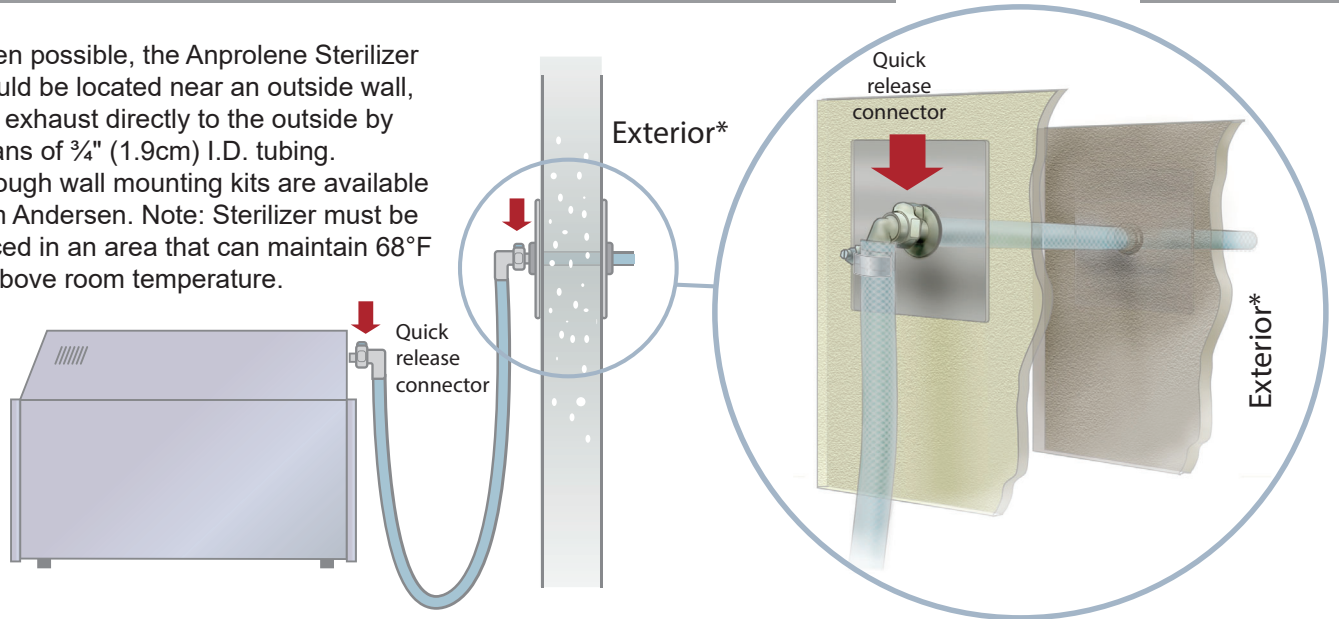

### Alternate Option 1 *Roof installation with vent boot*

Through the roof applications are best accomplished by installing 2" SCH 40 PVC using a roof vent boot and stabilizing the PVC to extend just below the ceiling. The  $\frac{3}{4}$ " I.D. to 2" adapter cap is available from Andersen. Note: 2" SCH 40 PVC exhaust can run up to 800' depending on path - bends/90° turns will reduce distance.

