


CaviWipes™

Step 1: Clean before disinfecting


1. Pull out CaviWipes towellette, tearing at a 90° angle.

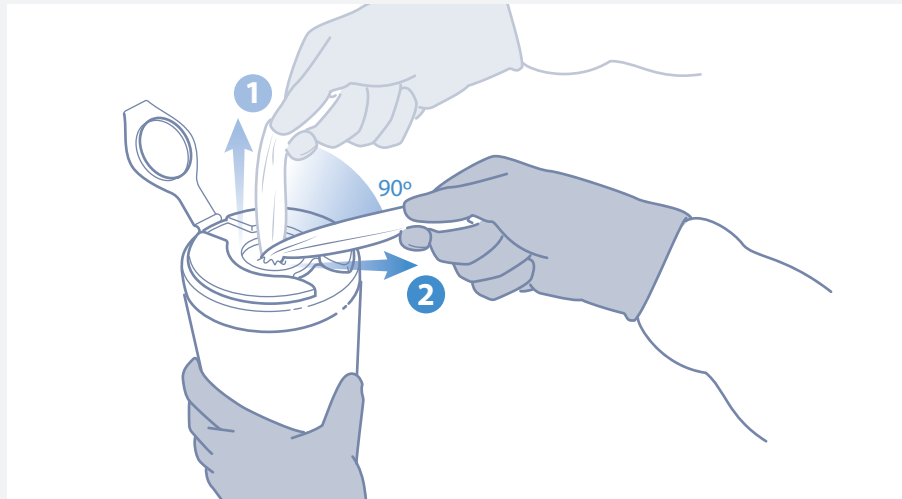

2. Use CaviWipes towelette(s) to completely pre-clean surface of all gross debris.



3. Dispose of used towelette in accordance with Federal, State, and local regulations for infectious materials disposal.

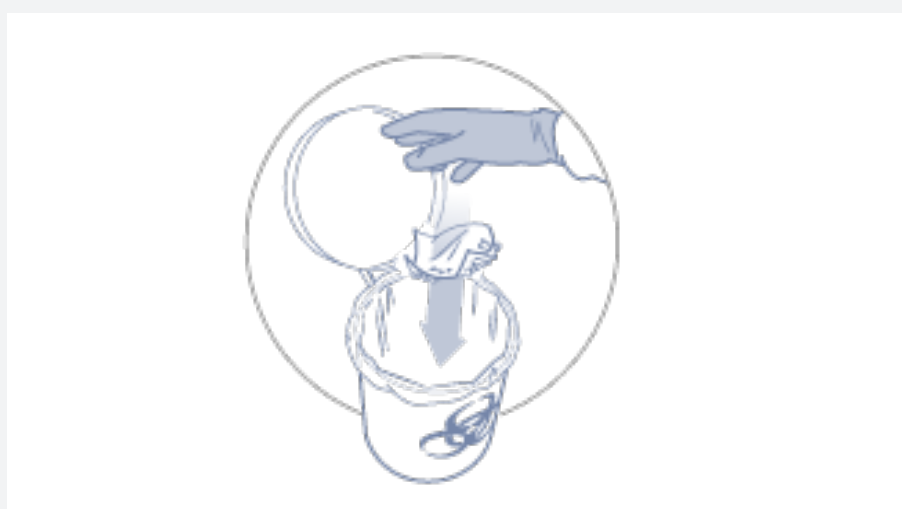

CaviWipes are intermediate level disinfectants recommended for use on hard, non-porous surfaces in healthcare settings.


- Effective against TB and MRSA in 3 minutes
- Effective against HIV-1, HBV, and HCV in 2 minutes
- For use on hard non-porous surfaces
- Cleaner and disinfectant

Step 2: Disinfect after cleaning

1. Pull out CaviWipes towellette, tearing at a 90° angle.



2. Disinfect surface with towelette. Leave surface wet for 3 minutes. Dispose of used towelette in accordance with Federal, State, and local regulations for infectious materials disposal.

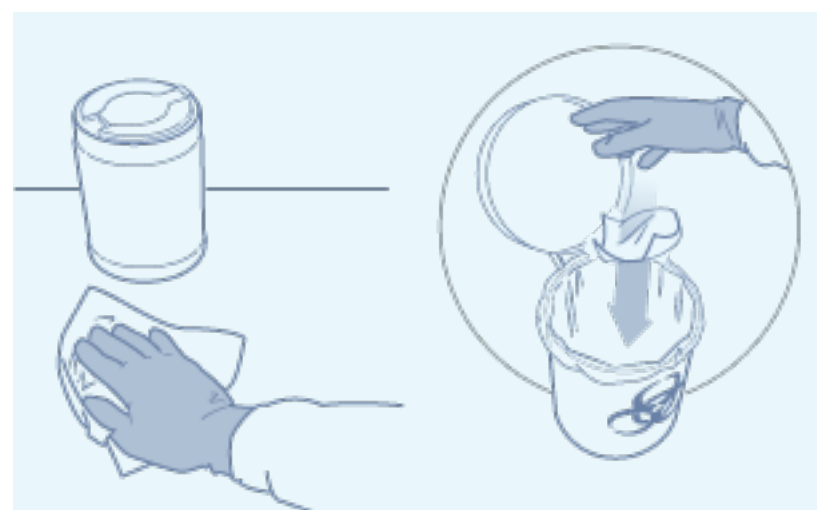

Ordering Information

| Part No. | Description | Packaging |
|----------|---|---------------------|
| 13-1100 | CaviWipes, 6" x 6 3/4" (canister of 160) | 12 canisters / case |
| 10-1090 | CaviWipes, 6" x 6 3/4" (canister of 220) | 12 canisters / case |
| 13-1150 | CaviWipes XL, 9" x 12" (canister of 65) | 12 canisters / case |
| 13-1155 | CaviWipes XL, 9" x 12" (box of 50) | 6 boxes / case |
| 13-1224 | CaviWipes, Flat Packs, 7" x 9" (pack of 45) | 20 packs / case |
| 13-1175 | CaviWipes canister wall bracket | 12 brackets / case |
| 13-1185 | Flat Pack Wall Mount | 20 mounts / case |

