

Result interpretation guide

Guía de interpretación de resultados

Guia de interpretação dos resultados

Chemdye® CD40

Unprocessed | Sin procesar | Sem processar



Failures | Fallas | Falhas



Processed and correct | Procesado y correcto | Processado e correto



Long Exposure |
Exposición prolongada |
Exposição longa

WARNING: Contact with Hydrogen Peroxide for times and concentrations greater than those declared can cause yellow tones.

ADVERTENCIA: El contacto con Peróxido de Hidrógeno por tiempos y concentraciones mayores a las declaradas puede causar tonos amarillos.

AVISO: O contato com Peróxido de Hidrogênio em tempos e concentrações maiores que as declaradas pode causar tons amarelos.

NOTE: Screen calibration of electronic devices (PC, tablet, mobile phone), as well as environmental light, may interfere in the representation of the colors. Terragene® does not guarantee that the color you see on the screen is 100% real.

NOTA: La calibración de la pantalla de los dispositivos electrónicos (PC/tablet/teléfono móvil), como así también la luz ambiental, pueden interferir en la representación de los colores. Terragene® no garantiza que el color que ve en pantalla sea 100% real.

NOTA: A calibração da tela dos dispositivos eletrônicos (PC/tablet/celular), assim como a luz ambiente, podem interferir na representação das cores. Terragene® não garante que a cor apreciada na tela seja 100% real.



DURALINE
BIOSYSTEMS INC

Clean
to Sterile
Solutions™

Authorized Terragene Distributor

Call (877) 561-0500 | sales@duralinesystems.com