



## Installation Guidelines EOGas 4 sterilizer installation options


## **Recommended Installation** When possible, the EOGas 4 Sterilizer Quick should be located near an outside wall, release connector and exhaust directly to the outside by Exterior\* means of 3/4" (1.9cm) I.D. tubing. Through wall mounting kits are available from Andersen. Note: Sterilizer must be placed in an area that can maintain 68°F or above room temperature. Exterior\* Quick release connector



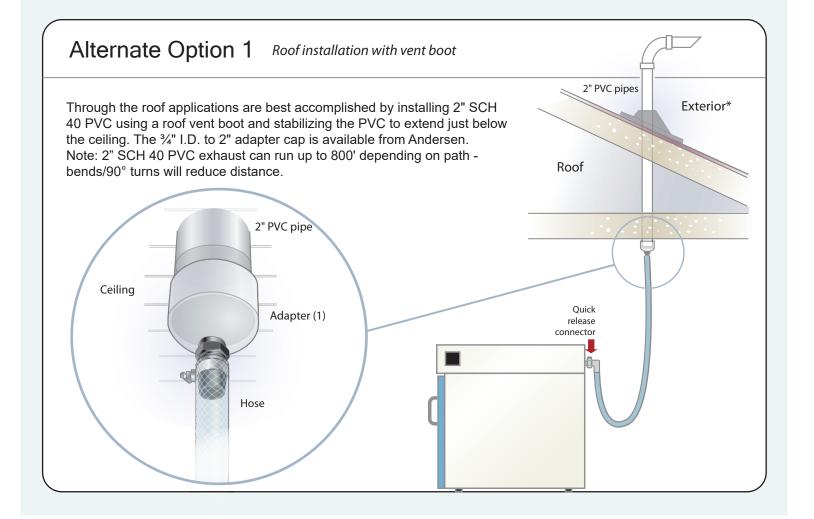

## **Installation Guidelines**

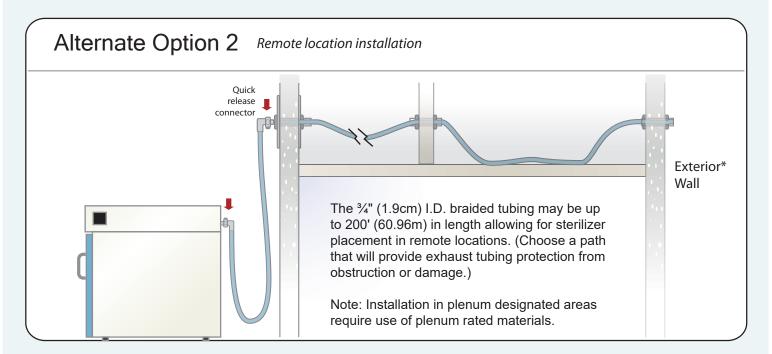

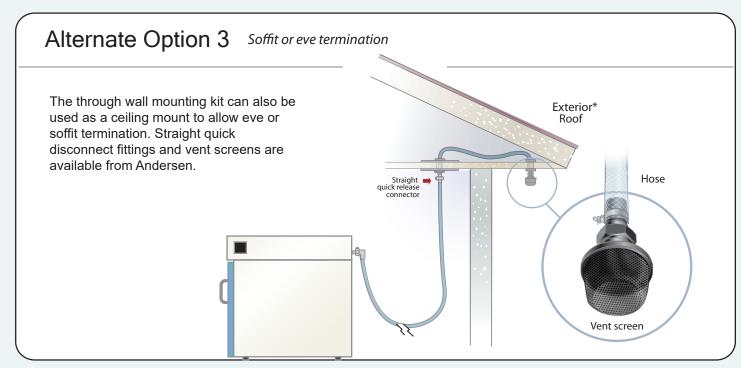

\* Please note: AAMI guidelines recommend all EO sterilizers be vented through a dedicated exhaut line to the outside atmosphere directly or through an emission control system. The vent should not terminate within 25 feet of any building air intake source or near pedestrian or animal traffic, nor should it allow EO release into or to reenter the building.

For further installation assistance please call your Andersen representative or 1.800.523.1276











Andersen Products, Inc.
Health Science Park • 3202 Caroline Drive
Haw River, NC 27258-9564 USA
800.523.1276



39 Western Hwy. West Nyack, NY 10994 www.DuralineSystems.com 877-561-0500