

DIRECTIONS FOR USE

INDICATIONS FOR USE

The Wash-Checks® Disposable Cleaning Monitor is designed for monitoring cleaning efficiency during the wash process.

INSTRUCTIONS FOR USE

1. Insert a Wash-Checks® monitor into a Wash-Checks® Holder completely. The front side of the monitor with red test soil will be visible in the holder window.



2. Position Wash-Checks® Holder face up inside a basket. Place basket on a shelf in the washer. Vary positions and shelves of Wash-Checks placement cycle to cycle.
3. Run a complete washer cycle. We recommend monitoring every shelf on every washer each day for a machine release and using a minimum of one Wash-Checks® per load for routine monitoring throughout the day.
4. At the end of the washing cycle, remove the Wash-Checks® monitor from the Holder and read the result. Caution: Wash-Checks® Holder and monitor strip may still be hot! Remove with care.

Monitor Results Interpretation:



Unexposed monitor or critical / poor result



Insufficient result



Optimum result

5. You may also record the location of monitor placement in the washer on the Wash-Checks® with a fine tip waterproof marker.
6. Staple the Wash-Checks® onto the Wash-Checks® Record Sheet and complete the appropriate information. Retain as a permanent record.

STORAGE

- Store Wash-Checks® monitors inside the protective foil bag provided at room temperature 50° - 100°F (10° - 38°C).
- Do not remove Wash-Checks® monitors from foil bag until ready for processing.
- Keep away from disinfectants and fumes.

EXPIRY DATE

The expiry date is printed on the product packaging.